

## ONETOUCH Reveal®



## OneTouch Reveal® web app


### Helps you get more meaning out of your results

- Tracks your blood glucose, food, insulin dosing and activity data over time
- Suggests personalized tips to help you stay on track and in range\*
- Reports can be printed, ready for discussion at your next appointment with your healthcare professional
- Can be used on PC and Mac computers at [OneTouchReveal.co.in](http://OneTouchReveal.co.in)

\* Be sure to talk to your healthcare professional about the actions that are right for you.

# How to setup your OneTouch Reveal® web app

Before you begin, confirm that cookies are enabled in your browser.

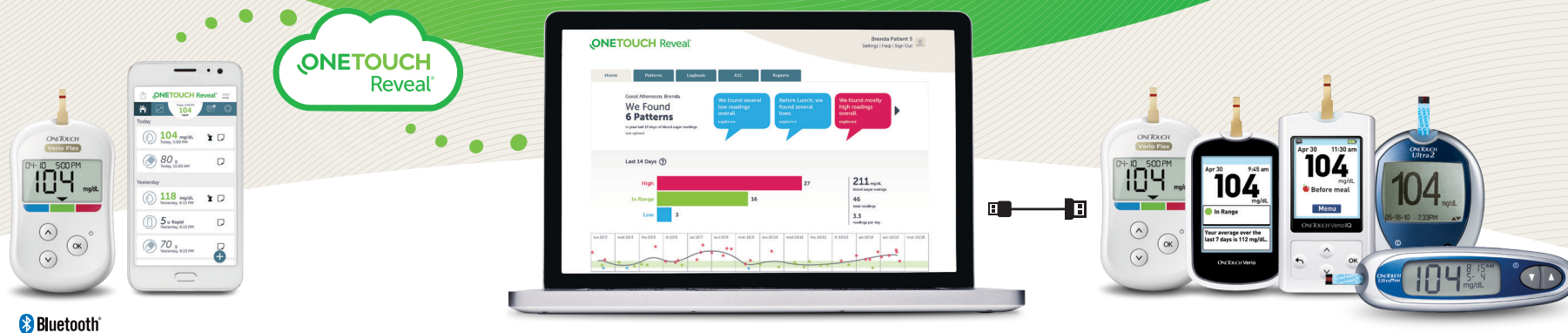

## Wireless Option

OR

## Cable Option

### 1. Connect wireless devices and activate your account

- Install the OneTouch Reveal® mobile app on your mobile device
- Open the mobile app and follow the on-screen instructions to setup your account and pair your OneTouch Verio Flex® meter
- Check your email inbox for a link to confirm your email address

For more details, refer to the OneTouch Reveal® mobile app **Help File**.

### 2. Sign in to the OneTouch Reveal® web app on your computer

- Go to **OneTouchReveal.com** and enter the same **Username** and **Password** that you used in the OneTouch Reveal® mobile app
- Click **Sign In** and follow the on-screen instructions

### 1. Register and activate your account

- Go to **OneTouchReveal.co.in** click on **Create a New Account** and follow the prompts
- Check your email inbox for a link to confirm your email address

### 2. Install Data Transfer Tool

- Unplug all OneTouch® USB cables from your computer
- **Sign In** to your OneTouch Reveal® account and click **help** from the top right of any screen
- Click the **Download** button under **Data Transfer Tool** and follow the prompts
- Select the appropriate devices
- Click **Finish**. After installation, you may be prompted to re-start your computer.


### 3. Transfer data from your OneTouch® meter

- Connect your meter to the computer with your OneTouch® USB cable (cable sold separately).
- Open the OneTouch Reveal® Data Transfer Tool and select **Detect Now**. When the device is found, a confirmation screen will appear.
- **Sign In** to the OneTouch Reveal® web application
- Click **Confirm** to transfer data. When the transfer completes, click **OK**




**You're all set! You can now access your data with the OneTouch Reveal® web app at [OneTouchReveal.co.in](http://OneTouchReveal.co.in)**

**ONETOUCH®**  
every touch is a step forward®

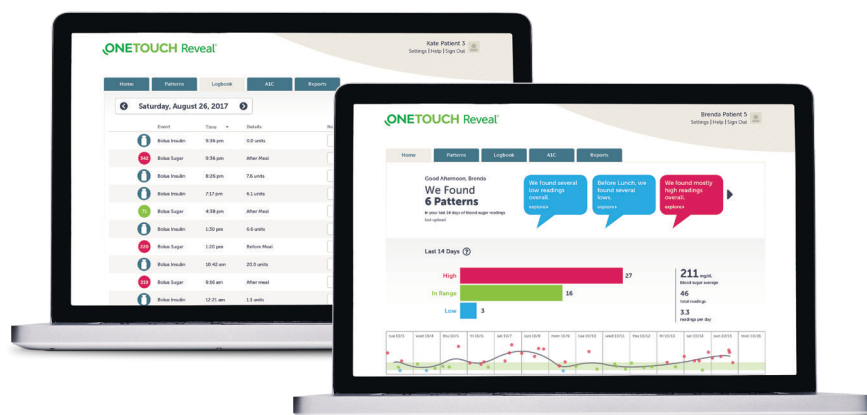

## View your blood glucose readings

- Click the **Logbook** tab to see your blood glucose readings
- Click the **Patterns** tab to view a 14-day summary

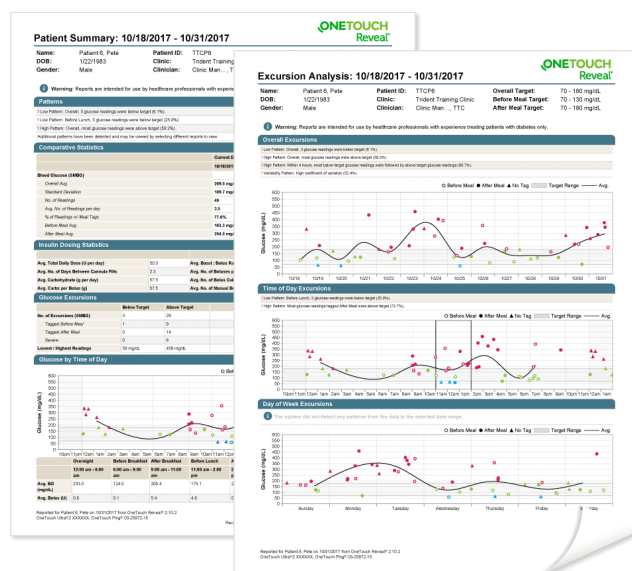

## Prepare for your next visit with your healthcare professional

- Click **Visiting your doctor?**  
The **Be prepared for your visit** screen will appear.
- Click **Download reports**  
To include reports, be sure to click the box next to **Include reports for your doctor** before you click **Download reports**.

If you'd like to change the list of reports you want to print for your visit, you can do so by clicking on **Settings**, then **Preferred Reports to Print** and then selecting the reports you need.

# Any questions? We're here to help



Visit **OneTouch.in**



Call us at 1 800-22-55-44



**ONETOUCH**<sup>®</sup>  
every touch is a step forward<sup>®</sup>