
Quick Start Guide

OneTouch Verio Flex™ Blood Glucose Monitoring System

Use this guide to get to know your meter, and follow the steps inside to get started.

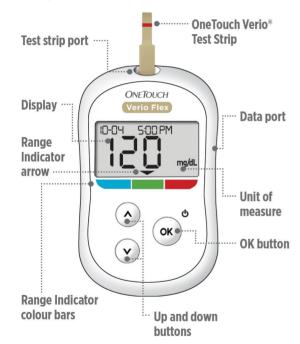

 \triangle IMPORTANT: Please read the Owner's Booklet that came with your system.



ONETOUCH®



Set the time and date on your meter

Turn it on 🖒

Press and hold ox until the start-up screen appears.

Use () buttons to edit each setting and press (or) to confirm.



Set the time on the first SET screen

- > **Hour** flashes. Press ox if correct or edit and confirm.
- > Minutes flash. Press (ox) if correct or edit and confirm.
- > AM/PM flashes. Press (ox) if correct or edit and confirm.

Set the date on the next SET screen

- > **Year** flashes. Press ox if correct or edit and confirm.
- > Month flashes. Press or if correct or edit and confirm.
- > **Day** flashes. Press ox if correct or edit and confirm.

Ready to test

Your meter setup is complete when the screen with 3 dashes appears.










2 Take a test

Silver prongs

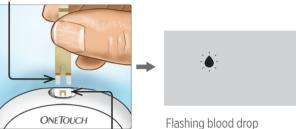


appears.


Insert a test strip (gold side and 2 silver prongs facing you) to turn the meter on.

Test strip port

Channel

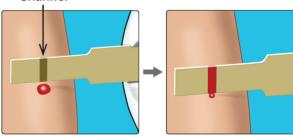


Apply blood sample to the channel on either side of the test strip. Wait for the channel to fill.

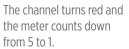

3

View your reading

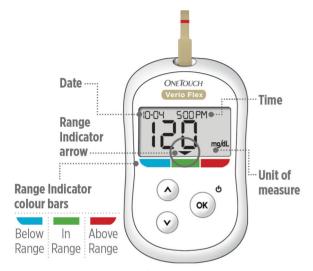

Example of an In Range reading

Your reading appears on the display. The Range Indicator arrow points to the blue, green, or red Range Indicator colour bar, to let you know your reading is below, within, or above your range limit.



Example of a Below Range reading



Example of an Above Range reading


4

Optional: Connect to a mobile device

Your meter comes with **Bluetooth®** Smart technology. This allows you to wirelessly send your readings to a mobile device such as a phone or a tablet, where you can view and graph them. To learn more about using your meter's wireless feature and view a complete listing of compatible devices and software applications, visit us at **www.onetouchdiabetes.co.in**.

Connecting your meter

Press on and of at the same time to turn the **Bluetooth**® feature on and off. When prompted by your mobile device, enter the PIN shown on your meter into your device.

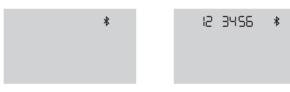

The ★ symbol means Bluetooth® is on.

Example of a PIN number.

Meter symbols Meter power Low battery Battery empty Control solution Syncing Meter power Bluetooth® feature on History Mode (past readings) Apply sample Range Indicator arrow SET Setting mode

About the Range Indicator

Your meter comes with a Range Indicator that automatically lets you know if your reading is below, above or within your range limits. The high and low range limits used by the Range Indicator are pre-set as follows:





Pre-set low range limit

Pre-set high range limit

For details about changing the pre-set range limits to suit your needs, refer to your Owner's Booklet.

The **Bluetooth**® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by LifeScan Scotland Ltd. is under license. Other trademarks and trade names are those of their respective owners.



Rev. Date: 06/2019 © 2015-2019 LifeScan, Inc. AW 07029902A Manufactured by: LifeScan Europe GmbH Gubelstrasse 34 6300 Zug Switzerland

Kindly contact Customer Service at 1800 225544 or visit



